

INSTRUCTIONS FOR REUSABLE EYE AND FACIAL PROTECTION

Cleaning and disinfection

Required Equipment


1 Cleaning disinfection products required:
Cleaning and disinfection wipes or solutions

2 PPE required:
Gloves

Reusable eye and facial protection (goggles, safety glasses, face shields without foam)



**If eye and facial protection appears damaged or compromised:
DO NOT REUSE**

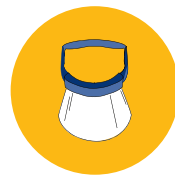

1. Don new pair of gloves.
2. Use a cleaning and disinfectant wipe or soap and water to clean and remove any visible soil.
3. Apply a disinfectant wipe or solution to all the interior followed by the exterior surfaces of the eye and facial protection.
4. Ensure all surfaces remain wet for the contact time specified on the disinfectant. Re-apply the disinfectant if it dries before the contact time.
5. If visibility is compromised by residual disinfectant (e.g., fogging), rinsing the eye and facial protection with tap water or wiping it with alcohol after disinfection may help.
6. Allow to dry (air dry or use clean absorbent towel).
7. Remove gloves and perform hand hygiene.
8. Store in a designated clean area.

Face shields with foam should not be reused unless directed by the organization. If reused, apply same cleaning and disinfection procedure (ensure disinfectant is applied to the interior, followed by foam band and strap and then the exterior surface). Dedicate the reused faceshield with foam to the same person.

2023.09.27