



# RESPIRATOR DONNING/DOFFING INSTRUCTIONS

The following instructions must be followed **each time** the respirator is worn.

## Donning procedures:

Before donning a respirator, perform hand hygiene and inspect the respirator to ensure the integrity of the component, including the shell, straps, and metal nose clip.

3M 1870+						
	<b>1.</b> Remove respirator from packaging and hold with straps facing upward. Place the bottom strap under the center flaps next to the <b>WARNING/ATTENTION</b> statement.	<b>2.</b> Fully open top and bottom panels, bending nosepiece around thumb at center of foam. Straps should separate when panels are opened. Make sure bottom panel is unfolded and completely opened.	<b>3.</b> Place respirator on face so that the foam rests on your nose and the bottom panel is under chin. Hold the bottom panel and pull the top strap, positioning it on the back of head.	<b>4.</b> Pull the bottom strap over your head and position it around the neck and below the ears. Adjust for a comfortable fit by pulling top panel toward the bridge of the nose and bottom panel under chin.	<b>5.</b> Place fingertips from both hands at the top of the metal nosepiece. Using two hands, mold the nose area to the shape of nose while moving fingertips down both sides of the nosepiece.	<b>6. USER SEAL CHECK:</b> Place both hands over the respirator and exhale gently. If air leaks are noted adjust facepiece and RECHECK. If a proper seal cannot be obtained, inform supervisor.

## Dooffing procedures:

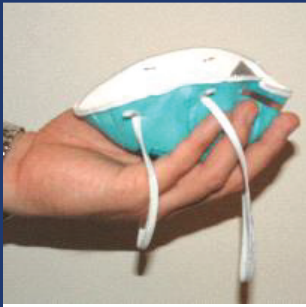
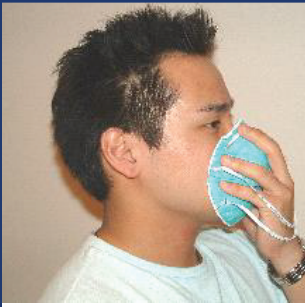




**(1)** Exit the patient room. **(2)** Perform hand hygiene. **(3)** Remove the respirator in the reverse order. Avoid touching the front of the respirator. Pull the bottom strap up and over head. Next, carefully pull the top strap up and over head thereby pulling the facepiece directly away from face and dispose in waste container. **(4)** Perform hand hygiene again.

# RESPIRATOR DONNING/DOFFING INSTRUCTIONS

The following instructions must be followed **each time** the respirator is worn.

## Donning procedures:

Before donning a respirator, perform hand hygiene and inspect the respirator to ensure the integrity of the components, including the shell, straps, and nose clip.

3M 1860 & 1860S (small)						
	<b>1.</b> Cup the respirator in hand with the nosepiece at fingertips, allowing the headbands to hang freely below hands.	<b>2.</b> Position the respirator under chin with the nosepiece up.	<b>3.</b> Pull the top strap over head so it rests high of the back of head.	<b>4.</b> Pull the bottom strap over head and bring it down below the ears. Position it around neck.	<b>5.</b> Using two hands, mold the nosepiece to the shape of nose by pushing inward while moving fingertips down both sides of the nosepiece.	<b>6. USER SEAL CHECK:</b> Place both hands over the respirator and exhale gently. If air leaks are noted adjust facepiece and RECHECK. If a proper seal cannot be obtained, inform your supervisor.

## Dooffing procedures:

**(1)** Exit patient room. **(2)** Perform hand hygiene. **(3)** Remove the respirator in the reverse order. Avoid touching the front of the respirator. Pull the bottom strap up and over head. Next, carefully pull the top strap up and over head thereby pulling the facepiece directly away from face and dispose in waste container. **(4)** Perform hand hygiene again.

# RESPIRATOR DONNING/DOFFING INSTRUCTIONS

The following instructions must be followed **each time** the respirator is worn.

## Donning procedures:

Before donning a respirator, perform hand hygiene and inspect the respirator to ensure the integrity of the components, including the shell, straps, and nose clip.

Halyard/Kimberly-Clark 46767

				
<b>1.</b> Separate the edges of the respirator to fully open it.	<b>2.</b> Slightly bend the nose wire to form a gentle curve.	<b>3.</b> Hold the respirator upside down to expose the two head bands.	<b>4.</b> Using index fingers and thumbs, separate the two head bands.	<b>5.</b> Holding the head bands with index fingers and thumbs, cup the respirator under chin.
				
<b>6.</b> Pull the head bands up over head.	<b>7.</b> Release the lower head band from thumbs and position it at the base of neck, beneath any hair.	<b>8.</b> Position the remaining head band on the crown of head.	<b>9.</b> Conform the nosepiece across the bridge of nose by firmly pressing down with fingers.	<b>10.</b> Continue to adjust the respirator and secure the edges until a good facial fit is obtained.
				<b>11. USER SEAL CHECK:</b> Forcefully inhale and exhale several times. The respirator should collapse slightly upon inhaling and expand upon exhaling. If any air leaking occurs between the face and the respirator adjust the respirator until the leakage is corrected. If a proper seal cannot be obtained, inform supervisor.

## Doffing procedures:

**(1)** Exit patient room. **(2)** Perform hand hygiene. **(3)** Remove the respirator in the reverse order. Avoid touching the front of the respirator. Pull the bottom strap up and over head. Next, carefully pull the top strap up and over head thereby pulling the facepiece directly away from face and dispose in waste container. **(4)** Perform hand hygiene again.